*The art of quality*

*A 600-years guarantee*

**I N T R A L I G A M E N T A L   S Y R I N G E S**

# Intraligamental syringes

Siringhe per anestesia intraligamentare • Jeringa-Intraligamentaria para anestesia • Seringues pour l'anesthésie intraligamental

## Advantages of Intraligamental Anesthesia:

- Automatic limitation of the power pressure.
- Anesthetizing of a single tooth.
- No numbness of the tongue, lips or cheek.
- Simultaneous treatment of teeth in different quadrants.
- Supports regional block anesthesia.
- Eliminates the risk of haematoma.
- Lower emotional impact on patient.

## Unique Medesyject Operating Features:

- Fully autoclaveable.
- Removable headpiece with angled threaded nozzle for the injection needle.
- Viewing chamber with safety sleeve for carpule cartridge.
- Handle servo dosing lever with optimal support for index finger for a gradual advance of the dosing plunger resetting key.
- A smaller quantity of given anaesthetic.
- No waiting time between injections and beginning of treatment.
- Excellent depth of anesthesia combined with shorter duration (45-55 minutes).
- Much less pain than standard injections (infiltration anesthesia in hard palate, intrapulpal or intraseptal injections or for papillectomy).

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Servo-dosing lever for optimal control and support for index finger.

Removable headpiece with angled threaded nozzle.

4964/AM  
4964/EU  
ml 1,8

Unique ratchet design for smooth delivery of anesthesia.

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• As opposed to a classical infiltration or block anesthesia, this method injects the local anesthesia directly into the dental ligament (periodontal ligament) and thus guarantees a rapid, profound and total anesthesia of a single tooth.

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## Syringes for intraligamental local anaesthesia. Advantages and indications.

The intraligamental technique facilitates treatments and saves time. Anaesthesia is profound and immediate and can be used for treatment of individual teeth. It avoids the palatal injections and eliminates the risk of haematoma. Automatic limitation of the power of pressure. Use is very simple: the needle is introduced into the gingival sulcus along the mesial or distal surface tooth until the alveolar bone crest is contacted. We recommend to inject very slowly, approx. 5-10 sec. per "click" with a non-excessive but constant pressure giving the analgesic solution the time to diffuse into the spongy bone. Contra-indications are for deep periodontal pockets and acute infections.

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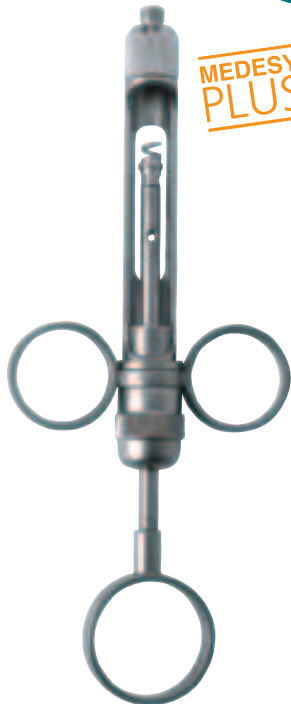
## EU - AM

= Availables with european EU and american thread AM.

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# Dental syringes

Siringhe dentali • Jeringas dentales • Seringues dentaires

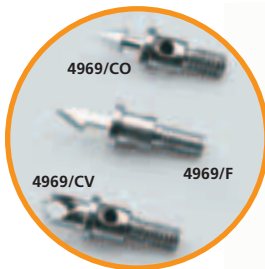
Our high quality dental syringes are manufactured in 18 versions. These are particularly suitable for the safe handling and emptying of phials and are equipped by 3 different designed tips which facilitate the penetration of the rubber phial stopper and ensure safe handling of phial as well as reliable extraction of the liquid. The syringes are packed with either european - EU - and american - AM - threads tips.

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4956, 4959, 4960, 4961, 4966.

• Save money and save the environment: don't waste the whole syringe; change the arrow, only (4969/CV-F-CO).

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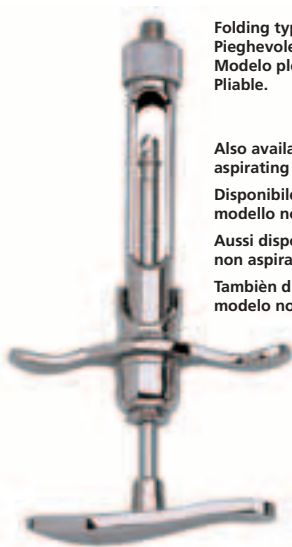
**4968**  
ml 1,8  
EU

## Passive Automatic Aspiration

Aspiration, before and during the injection of anaesthetic, is strongly recommended by the conventional literature of dental surgery. The development of passive systems which depend on a "semi automatic" backward movement of the components of the cartridge made easier the process of aspiration. A well-balanced syringe which suits comfortably to the hands gives to the dentist good tactile informations transferred from the tip of the needle to the fingertip of the syringe. This should be associated with user-friendly aspiration system. This two elements must be the reason of the choice of an injection instrument.

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### The syringe offers two simple ways to achieve easily the aspiration:

1. Aspiration is provided either by gentle releasing the pushing force on the piston - in this way the elastic membrane in the cartridge, previously deformed by the pushing, it comes back to its original shape ensuring an aspiration effect -
2. or by pressing and releasing the thumb disk.

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